Delton® Pit & Fissure Sealant
Delton® FS+ Flowable Sealant with Fluoride
Syringe Delivery System

Directions for Use - English

Caution: United States Federal law restricts this device for sale by, or on the order of, a licensed dental professional.

Delton® FS+ provides a quick and simple technique for applying sealant material to the teeth.

Delton® FS+ is an opaque or tooth colored, 55% glass filled pit and fissure sealant with releasable fluoride.

COMPOSITION:

Delton® FS+ Pit and Fissure Sealant:

- Low viscosity monomers
- Triethylene glycol dimethacrylate
- BisGMA
- Barium alumino fluroboro silicate glass
- Titanium dioxide (opaque only)
- Sodium Fluoride
- Polymerization initiator
- Stabilizer

INDICATIONS FOR USE: Preventive sealing of pits and fissures in the primary and secondary dentition in combination with the acid-etch technique.

CONTRAINDICATIONS: Known allergies to any components. Not to be used in case of caries.

WARNINGS:

Delton® FS+ Pit and Fissure Sealant:

Contains methylacrylates which may be irritating to the skin and eyes. Avoid contact with oral tissues, eyes and skin. If accidental contact with eyes occur, flush affected area with generous amounts of water and seek medical attention. If accidental contact with skin occurs, wash immediately with plenty of soap and water. The product may cause sensitization by skin contact in susceptible persons. If skin sensitization occurs discontinue use.

PRECAUTIONS: Delton® FS+ Pit and Fissure Sealant syringes should be tightly closed immediately after use. Keep Delton® FS+ Pit and Fissure Sealant out of direct sunlight. Recommended storage; do not exceed 24° C (75° F). Recommended usage; use at room temperature.

ADVERSE REACTIONS: Since reversible inflammatory changes of the oral mucosa may occur after accidental contact with acrylate monomers, avoid contact of Delton® with mucous membranes.

Anaphylactic reactions may occur in susceptible individuals. If patient shows signs of anaphylaxis, seek medical attention immediately.

STEP-BY-STEP INSTRUCTIONS:

Cleaning:

Clean surfaces to be sealed with an air-prophylaxis device (Cavitron® Prophyl-Jet or Cavitron® Jet Plus) or an oil-free paste (Nupro® Prophyl Paste) and a prophyl angle. Rinse well with water.

Isolating:

Isolate the teeth to be sealed with a rubber dam or cotton rolls and dry each tooth with air, free of oil or water contamination.

Etching:

Application of Delton® EZ Etch Etchant Gel:

NOTE: Apply a disposable barrier (Disposa-Shield®) to syringe to reduce cross-contamination between patients and dental practitioners. See Directions For Use for Disposa-Shield® Disposable Polyethylene Barriers for Etchant or Sealant Syringes. Follow instructions detailed in the EZ Etch 34% Etchant Gel Directions for Use.

Sealing:

Use at room temperature.

Application of Delton® FS+ Pit and Fissure Sealant:

NOTE: Apply a disposable barrier (Disposa-Shield®) to syringe to reduce cross-contamination between patients and dental practitioners. See Directions For Use for Disposa-Shield® Disposable Polyethylene Barriers for Etchant or Sealant Syringes. Remove the cap from the Delton® FS+ syringe. Attach a disposable brush tip applicator to end of the syringe. Turn tip clockwise 1/4 to 1/2 turn to assure that it is fully seated. Tug on the tip to be sure that it is locked into the collar of the syringe. To assure free flow of material from syringe, express a small amount onto a pad. Apply sealant into the pits and fissures through brush tip applicator.

Curing:

Cure each surface for at least 20 seconds with a polymerization unit keeping the light exit window as near as possible to the tooth without touching it. Remove soft (oxygen-inhibited) surface layer after light-curing with cotton pellets or cotton rolls.

Retention and Occlusion Control:

Check sealed area with an explorer for complete coverage and retention. Check occlusion with articulating paper and adjust if necessary with a finishing bur.

Syringe Care:

Delton® FS+ Pit and Fissure Sealant syringe and Delton® EZ Etch Etchant Gel syringe care.

NOTE: Remove disposable barrier (Disposa-Shield) from syringe. See Directions For Use for Disposa-Shield Disposable Polyethylene Barriers for Etchant or Sealant Syringes. Carefully remove the spent brush tip applicator from the syringe and discard appropriately. Replace each cap tightly. Wipe the entire syringe with disinfectant and dry with clean cloth.

STORAGE: Syringes should be closed immediately after use. Keep out of direct sunlight, do not store at temperatures exceeding 24° C (75° F).

BATCH NUMBER AND EXPIRY DATE: Do not use after expiry date.

The batch number indicates important production information. The number should be quoted in all correspondence which requires identification of the product.

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