Fusio™

Self-Adhesive Flowable Composite

English

INSTRUCTIONS FOR USE

Français

French

MODE D'EMPLOI

Deutsch

German

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Italiano

Italian

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Español

Spanish

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Čeština

Czech

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English

INSTRUCTIONS FOR USE:

Indications:

- Self-Adhesive base liner under higher viscosity restorative materials
- Class III restorations
- Class V restorations
- Restoration of small Class I cavities
- Blocking out undercuts
- Pit and fissure sealant

Contraindications:

- Avoid placing Fusio Liquid Dentin in stress bearing areas. Use Artiste™ Nano Composite for these applications.
- Do not apply phosphoric acid etch to dentin prior to placing Fusio Liquid Dentin.
- Not recommended for direct pulp capping.

Caution: The use of this product should be avoided on patients with a known sensitivity to any of its ingredients.

Incompatibility:

- Avoid using materials that contain eugenol or clove oil as they may inhibit the polymerization of this composite and interfere with bonding to tooth structure. Contact with cationic mouth rinses, plaque disclosing agents and chlorhexidine may result in discoloration.

GENERAL INSTRUCTIONS FOR USE

(Class I, III, V, blocking out undercuts):

1. Clean the teeth before selecting the shade.
2. Isolate the treatment area with a rubber dam.

Important Note for Class V Preparation with Caries: Ensure there is a physically present and well defined cavity preparation. Create an outline form removing all caries at DEJ and all decalcified enamel. Place a bevel on all enamel margins.

3. Prepare the cavity according to the principles of adhesive dentistry. Avoid any sharp internal edges. In anterior regions, bevel the enamel edges for better retention. Thoroughly clean the cavity and remove any potential debris with water. Blot or air-dry briefly (2–3 seconds) to remove excess water for a moist surface with no pooling water. The surface should glisten and appear slightly shiny. Do not desiccate the tooth.

Important Note: Ensure that surface debris is removed and the preparation is clean. Intimate

contact of Fusio Liquid Dentin to the tooth surface is essential for achieving a good and reliable bond.

4. For deep cavities cover the floor adjacent to the pulp with calcium hydroxide material first and subsequently use Fusio Liquid Dentin.
5. Use a transparent matrix for cavities affecting the proximal areas. In the posterior region, a stainless steel matrix may be used with caution as the flowable may adhere to the matrix band.
6. Using the bent needle tip supplied, dispense a 1mm increment of Fusio Liquid Dentin directly onto the tooth surface. Gently rub the material into the preparation using the needle tip for 20 seconds. A Microbrush™ micro applicator may also be used to agitate the initial layer of Fusio Liquid Dentin for 20 seconds. Pull the syringe plunger back after use to prevent overflow of the Fusio Liquid Dentin from the syringe tip.
7. Light cure the initial layer for 10 seconds with a curing light with output in excess of 600 mW/cm².
8. Syringe additional Fusio Liquid Dentin as necessary. Do not apply in increments of more than 2mm thickness for proper light curing for the subsequent layer(s). Light cure each additional 2mm increment for 10 seconds. The final layer should be cured for an additional 10 seconds.
9. Remove excess material with a Fini™ Polishing Disk or a fine grain diamond. Remove the proximal excess with diamond or tungsten carbide finishers or finishing strips. Check the occlusion and adjust it as necessary.
10. Polish the restoration to a high gloss using Fini Polishing disks.
11. For maximum surface sealing, to reduce surface wear, and to minimize microleakage at the bonding interfaces, following trimming and polishing apply Protect-It™ Composite Surface Sealant to seal the restoration surface and surrounding tooth surface margins.
(Please see the Protect-It™ Composite Surface Sealant instructions for step by step details.)

PIT AND FISSURE INSTRUCTIONS FOR USE:

1. Isolate the treatment area with a rubber dam.
2. Prepare the enamel surface with an air abrasion unit following the manufacturer's instructions. If an air abrasion unit is not available use 37% phosphoric acid.
3. Apply 37% Phosphoric Acid Etching Gel* to enamel and dentin surfaces for 15 seconds. Rinse thoroughly for 15 seconds with water. It is important not to splash etch into eyes, onto soft tissues or skin.
4. Dry with clean, oil-free, air without desiccating the

dentin. Do not allow etched surface to become contaminated with saliva or blood during the bonding procedure. NOTE: Never use excessive force on syringe plunger to dispense etch which may lead to over dispensing of the material. If resistance is felt replace current tip with a new tip and check flow extraorally.

5. Using the bent needle tip supplied, gently rub the material into the preparation using the needle tip for 20 seconds while dispensing Fusio Liquid Dentin directly onto the tooth surface. A Microbrush micro applicator may also be used to agitate the Fusio Liquid Dentin for 20 seconds. Pull the syringe plunger back after use to prevent overflow of the Fusio Liquid Dentin from the syringe tip.
6. Light cure for 20 seconds with a curing light with output in excess of 600 mW/cm².
7. Check the occlusion and adjust it as necessary.

DENTIN REPLACEMENT MATERIAL OR SELF-ADHESIVE BASE LINER INSTRUCTIONS FOR USE

1. Clean the teeth before selecting the shade.
2. Isolate the treatment area with a rubber dam.
3. Prepare the cavity according to the principles of adhesive dentistry. Avoid any sharp internal edges. Remove any potential debris with water and thoroughly clean the cavity. Blot or air-dry briefly (2-3 seconds) to remove excess water and form a moist surface with no water pooling. The surface should glisten and appear slightly shiny, not dull. Do not desiccate the tooth.

Important Note: Ensure that surface debris is removed and the preparation is clean. Intimate contact between Fusio Liquid Dentin and the tooth surface is essential for achieving a good and reliable bond.

4. For deep cavities cover the floor adjacent to the pulp with calcium hydroxide material and subsequently use Fusio Liquid Dentin.
5. Use a transparent matrix for cavities affecting the proximal areas. In the posterior region, a stainless steel matrix may be used with caution as the flowable may adhere to the matrix band.
6. Using the bent needle tip supplied, dispense a 1mm increment of Fusio Liquid Dentin directly onto the tooth surface. Gently rub the material into the preparation using the needle tip for 20 seconds. A Microbrush micro applicator may also be used to agitate the initial layer of Fusio Liquid Dentin for 20 seconds. Pull the syringe plunger back after use to prevent overflow of the Fusio Liquid Dentin from the syringe tip.
7. Light cure the initial layer for 10 seconds with a

- curing light with output in excess of 600 mW/cm².
8. Continue to add subsequent layers of Fusio Liquid Dentin as desired. Do not apply in increments of more than 2mm thickness for proper light curing for the subsequent layer(s). Light cure each additional 2mm increment for 10 seconds. The final layer should be cured for an additional 10 seconds.
 9. Total etch the enamel, any dentin that may have not been covered by the Liquid Dentin, and the Fusio Liquid Dentin filled prep with 37% Phosphoric Acid Etching Gel.
 10. Apply 37% Phosphoric Acid Etching Gel* to enamel and dentin surfaces for 15 seconds. Rinse thoroughly for 15 seconds with water. It is important not to splash etch into eyes, onto soft tissues or skin.
 11. Dry with clean, oil-free, air without desiccating the dentin. Do not allow etched surface to become contaminated with saliva or blood during the bonding procedure. NOTE: Never use excessive force on syringe plunger to dispense etch which may lead to over dispensing of the material. If resistance is felt replace current tip with a new tip and check flow extrorally.
 12. Apply 2 coats of Bond-1™ Primer/Adhesive to the enamel and flowable composite. Lightly air thin for 10 seconds to remove the solvent. Light cure for 10 seconds.
 13. Apply an Artiste Nano Composite enamel shade in 0.5mm to 2mm layers maximum, cure, and polish.

Microbrush is a registered trademark of Microbrush International.

NOTE: Tips and applicator brushes are for single patient use only, to prevent cross-contamination between patients.

NOTE: Refer to outer package for expiration date. Do not use after expiration date.

***CAUTION:** 37% Phosphoric Acid Etching Gel contains phosphoric acid. Avoid contact with skin, eyes, and soft tissue. In case of contact with skin or eyes, flush immediately with water; get medical attention for eyes. Do not take internally.

CAUTION: Bond-1 Primer/ Adhesive contains acetone and is flammable and volatile. Keep away from sources of ignition and excessive heat.

CAUTION: Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact

with skin, eyes and soft tissue. Wash thoroughly with water after contact.

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Self-Adhesive Flowable Composite

Rx Only

Refrigeration required.
Store at 2-8°C. (36-46°F)

LIMITED WARRANTY —

Pentron's technical advice, whether verbal or in writing, is designed to assist dentists in using Pentron's product. The dentist assumes all risk and liability for damages arising out of the improper use of Pentron's product.

In the event of a defect in material or workmanship, Pentron will replace the defective product or part thereof, or reimburse the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Pentron.

PENTRON

Manufactured by Pentron Clinical

1717 West Collins Ave. Orange, CA 92867 USA
800-551-0283 / 714-516-7557 / www.pentron.com

EC REP European Union Representative
SpofaDental a.s.
Markova 238
CZ-506 01 Jičín
Phone: +420 493 583 111