**LIGHT-CURED COMPOSITE RESIN****[ENGLISH] INSTRUCTIONS FOR USE****I. INTRODUCTION**

CLEARFIL PHOTO POSTERIOR is a light-curing restorative composite for posterior teeth. A range of 6 shades permits the construction of esthetic and durable posterior restorations.

II. INDICATIONS

Composite resin restoration for posterior tooth; Class I, II and V cavities

Patients with a history of hypersensitivity to methacrylate monomers

III. CONTRAINDICATIONS

Contains a history of hypersensitivity to methacrylate monomers

IV. INCAPABILITY

Do not use eugenol-containing materials for pulp protection or temporary sealing since the eugenol can retard the curing process.

V. PRECAUTIONS

5.1. Safety precautions
The product contains substances that may cause allergic reactions.

Use of the product in patients with known allergies to methacrylate monomers or any other components.

If the patient demonstrates any allergic reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching or numbness, discontinue the use of the product and seek medical attention.

Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers or any other components.

Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.

If the product comes in contact with human body tissues, take the following actions:

• If the product gets in the eyes: Immediately wash the eye with copious amounts of water and consult a physician.

• If the product comes in contact with the skin or the oral mucosa: Immediately wipe the area with a cotton pad/pledge or a gauze moistened with alcohol, and rinse with copious amounts of water.

Use caution to prevent the patient from accidentally swallowing the product.

Avoid looking directly at the dental curing light when curing the product.

Avoid sharing the same paste among different patients to prevent cross contamination.

Dispense the paste on a mixing paper before use.

Do not try to apply the paste directly from the syringe onto the patient's tooth.

6. Handling and manipulation precautions

The use of this product is restricted to licensed dental professionals.

The product should be returned to room temperature before dispensing. If it has been taken out of a refrigerator, failure to do this might cause breakage of the syringe.

Use the composite within 5 minutes after dispensing. Exposure to the operating light will cause the composite to cure.

Do not use the product in conjunction with other composite resin.

The mixed materials may cause a change in physical properties, possibly a decrease, from the properties expected.

When light curing the product, note the depth of cure in this Instructions for Use.

The emitting tip of the dental curing unit should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.

Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light type for contamination. It is advisable to check the dental curing light intensity using an appropriate light-evaluating device at appropriate intervals.

7. Storage precautions

The product must be used by the expiration date indicated on the package.

The product must be stored at 2 - 25°C 36 - 77°F when not in use.

After the paste has been dispensed, turn the plunger counter-clockwise a half-turn to prevent excess composite from oozing out. Then, the syringe should be capped securely as soon as possible to prevent the paste inside the syringe from being hardened by ambient light, and to keep foreign matter from entering the syringe.

The product must be stored in a proper place where only dental practitioners can access it.

VI. SHADES SYSTEM AND COMPONENTS**1. Shades**

CLEARFIL PHOTO POSTERIOR is available in the following 6 shades: US, UL, XL, UY, B and G

2. Components

Please see the outside of the package for contents and quantity.

3. Ingredients

Principal ingredients
• Silanated silica filler
• Silanated barium glass filler
• Silanated colloidal silica

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