DENTAL LIGHT-CURED OPAQUE RESIN

CLEARFIL™ ST OPAQUER

ENGLISH INSTRUCTIONS FOR USE

I. INTRODUCTION

CLEARFIL ST OPAQUER is a light-cured composite resin used to mask underlying metal darkness and tooth discoloration.

II. INDICATIONS

- Masking of metal during intraoral repair of porcelain fused-to-metal crowns
- Masking of metal in restoration of post and core crowns
- Lightening of stained or discolored teeth in porcelain or composite veneer restoration
- Masking of pulp capping materials

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to methacrylate monomers

IV. INCOMPATIBILITIES

Do not use eugenol containing materials for pulp protection or temporary sealing, since the eugenol can retard the bonding system curing process.

V. PRECAUTIONS

- [Safety precautions]
- If any hypersensitivity such as dermatitis occurs, discontinue the use of the product and consult a physician.
 - Wear gloves or take other appropriate protective measures to prevent the occurrence of hypersensitivity that may result from contact with methacrylate monomers.
 - Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect the patient's eyes from splashing materials.
 - If the product comes in contact with human body tissues, take the following actions:
 - < If the product gets in the eye > Immediately rinse the eye with copious amounts of water and consult a physician.
 - < If the product comes in contact with the skin or the oral mucosa > Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and rinse with copious amounts of water.
 - Use caution to prevent the patient from accidentally swallowing the product.
 - Do not look directly at the light source. Protective glasses are recommended.
 - To prevent cross infection, avoid sharing the same paste among different patients.

[Handling and manipulation precautions]

- The use of the product is restricted to licensed dentist.
- Provide moisture and contamination control with the use of a rubber dam.
- Any exposed pulp or areas near the pulp should be covered in a correct manner, such as pulp capping.
- Do not use the product for purposes other than listed in INDICATIONS.
- Make sure to wipe any remaining paste off the tip of the syringe with a cotton pledget before closing the cap; otherwise the cap might come off.
- Amalgam or other lining materials remaining in the cavity will prevent the passage of light and the polymerization of the product. Completely remove any lining material when preparing the cavity.
- When light curing the product, note the light curing depth in this Instructions for Use.
- The emitting tip of the dental curing light should be held as near and vertical to the resin surface as possible. If a large resin surface is to be light cured, it is advisable to divide the area into several sections and light-cure each section separately.
- Low light intensity causes poor adhesion. Check the lamp for service life and the dental curing light guide tip for contamination. It is advisable to check the dental curing light intensity using an appropriate light-evaluating device at appropriate intervals.

[Storage precautions]

- The product must be used by the expiration date indicated on the package.
- The product must be stored at 2 - 25°C/ 36 - 77°F when not in use.
- Keep away from extreme heat and direct sunlight.
- The product must be stored in proper places where only dental practitioners can access it.

VI. COMPONENTS

1. Shades

CLEARFIL ST OPAQUER is available in 2 shades; US and L

2. Components

Please see the outside of the package for contents and quantity.

3. Ingredients

- Principal ingredients
- Silica filler
 - Silanated colloidal silica
 - Colloidal silica
 - Triethyleneglycol dimethacrylate
 - Bisphenol A diglycidylmethacrylate
 - dl-Camphorquinone

VII. CLINICAL PROCEDURES

1. Adherent treatment

- 1-1. Pretreatment of tooth surface
After preparing the tooth surface, apply a phosphoric acid (e.g. K-ETCHANT GEL) according to the manufacturer's instructions.

1-2. Pretreatment of metal surface

Roughen the metal surface by sandblasting with alumina powder or using a diamond point, before washing and drying. Then, if necessary, apply a metal-adhesive primer (e.g. ALLOY PRIMER) according to the Instructions for Use of the product used.

1-3. Pretreatment of porcelain and composite resin

Remove a layer of the fractured surfaces and place a bevel at the marginal area using suitable instruments. (e.g. diamond point)
Then, apply a phosphoric acid etching agent (e.g. K-ETCHANT GEL) to the adherent surface and leave it in place for 5 seconds before washing and drying.
After that, apply a silane-coupling agent (e.g. CLEARFIL CERAMIC PRIMER) according to the Instructions for Use of the product used.

2. Bonding

Bonding should be performed according to instructions for Use of the bonding system used (e.g. CLEARFIL SE BOND).

3. Applying the opaquer

Apply the product with a thickness not exceeding 0.5 mm to the adherent surface, and then light cure the paste with a dental curing unit following the table.

Table: Relation between Light curing time and thickness of cure for each dental curing unit

Dental curing unit	Thickness and curing time	
	less than 0.4 mm	0.4 to 0.5 mm
Conventional halogen*	40 seconds	60 seconds
Fast halogen*	10 seconds	20 seconds
Plasma arc*	5 seconds	5 seconds
LED*	40 seconds	80 seconds

*Dental curing unit

Type	Light source	Range of wavelength and light intensity
Conventional halogen	Halogen lamp	Light intensity ³⁾ of 300-550 mW/cm ² with wavelength range of 400-515 nm
Fast halogen	Halogen lamp	Light intensity ³⁾ over 550mW/cm ² wavelength range from 400-515 nm
Plasma arc	Xenon lamp	Light intensity ³⁾ over 2000mW/cm ² in wavelength range of 400-515 nm, and light intensity over 450 mW/cm ² in wavelength range of 400-430 nm
LED	Blue LED ¹⁾	Light intensity ³⁾ over 300mW/cm ² in wavelength range from 400-515 nm

- Peak of emissions spectrum: 450-480nm
- Evaluated according to ISO 10650-1.
- Wavelength distribution and light intensity values measured with a spectroradiometer calibrated using an IEC or NIST (National Institute of Standards and Technology) standard lamp
- Evaluated according to ISO 10650-2.

[NOTE]

If the product is to be applied to make a layer thicker than 0.5 mm or if it is to be applied in areas difficult to reach with the light, apply and cure it in a few divided sessions.

4. Light-curing composite filling

Apply composite resin (e.g. CLEARFIL AP-X) into the cavity, light-cure, finish and polish according to the manufacturer's instructions.

[CAUTION]

Federal (U.S.A.) law restricts this device to sale by or on the order of a licensed dentist.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective.
Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

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"CLEARFIL", "CLEARFIL AP-X", "CLEARFIL ST" and "CLEARFIL SE BOND" are trademarks of KURARAY CO., LTD.