VOCO

ES

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Fluoridin N 5

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Instructions for use

Fluoridin N 5

active ingredient: sodium fluoride

for caries prophylaxis

Fluoridin N 5 is a pleasant tasting, fruit flavoured fluoridation gel which turns to liquid after application, allowing effective penetration of interdental areas. **Fluoridin N 5** tolerates residual moisture and thus can be applied to areas difficult to dry.

- Indications:**
- General caries prophylaxis for children, adolescents and adults
 - Initial caries treatment support
 - Treatment areas prone to caries such as discoloured fissures and/or carious decalcification during orthodontic adjustments

Side effects:
None reported to date.
Hypersensitive patients may experience nausea and/or gum irritation when exposed to excessive amounts of **Fluoridin N 5**.

Application:
Thoroughly clean and remove any tartar from the areas to treated and dry, if possible.

a. By brush:
Apply **Fluoridin N 5** with a brush or foam pellet to high caries risk areas, such as fissures and/or approximal areas.

b. Direct application with a cartridge:
Always use a blunt syringe tip for paste materials (length 32 mm, diameter 0.9 mm). Apply **Fluoridin N 5** in thin layers directly onto the targeted area. Bend the tip with pliers or by hand to access approximal areas. Replace with a new tip before the next application.

Allow **Fluoridin N 5** to dry or penetrate.
Do not brush the teeth for 24 h after treatment as residual varnish may be removed by brushing.

Frequency of application:
For prophylaxis, treatment should be carried out 2 - 3 times a year or 3 - 4 times a year for greater caries susceptibility.

Additional instructions:
Fluoridin N 5 cartridges can be used several times. Remove the tip after application and store the cartridge without a tip (The cartridge is tightly sealed by the rubber). Also store **Fluoridin N 5** syringes without a tip and close with a rubber cap. Attach a new tip for every application. Use only blunt syringe tips. Application methods involving larger quantities, such as tray application, are not suitable. Do not use after the expiry date. Do not store at temperatures above 25° C. Store in safe places which cannot be reached by children.

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VOCO GmbH