x-tra fil

In accordance with DIN EN ISO 4049

Carefully read instructions prior to use

x-tra fil is a light-curing, highly radiopaque hybrid composite designed specifically for time-saving cost-effective use in the posterior region. The combination of a new multi-hybrid filler technology with an innovative initiator system formed the basis for a filler material exhibiting minimal polymerization shrinkage and excellent depth of cure. **x-tra fil** can even be cured reliably in increments of 4 mm and with very short polymerization times. **x-tra fil** undergoes reduced shrinkage stress to ensure restorations with integral margins. The physical strength and high abrasion-resistance are high enough for occlusal load-bearing fillings. **x-tra fil** contains 86% by weight inorganic filler (= 70.1% by volume). **x-tra fil** comes in a universal shade for posterior restorations (U). **x-tra fil** is cured with halogen or LED light with a blue spectrum and is available in practical syringes or caps for direct application.

Indications:

- Class I and II posterior filling
- Core build-up

Contraindications:

x-tra fil should not be used if the recommended application technique cannot be used or if the patient has a known allergy to any of the ingredients (BIS-GMA, UDMA, BHT, TEDMA).

Methods of use:

Preparation

Clean the teeth to be treated. Mark occlusal contact points - minimal separation facilitates the design of the proximal contact and the placing of the matrix. Using a rubber dam is recommended.

Cavity preparation

Generally, cavity preparation should be carried out according to the rules of the adhesive filling therapy, i.e. minimally invasive to preserve healthy tooth substance. Clean and dry the cavity.

Lining

A calcium hydroxide lining should be applied in the proximity of the pulp. Place a layer of stable cement over the lining. Due to their fluoride release, glass polyalkenoate (glass ionomer) materials are recommended.

Bond material

Follow the special requirements for bonding agents (see the relevant instructions). **x-tra fil** is used in adhesive technique with a dentin/enamel bond. All light-curing bonding materials can be used. Follow the respective instructions for use with regards to preparation and application. The use of **Futurabond NR** is recommended. Follow the instructions for use of **Futurabond NR**.

Application of x-tra fil

Prior to use, allow the material to attain room temperature. Place a matrix on cavities in the approximal area. The use of translucent matrices is advantageous (smooth surfaces). Apply **x-tra fil** and adapt it with a suitable instrument.

x-tra fil Caps

Insert caps into the opening of the dispenser (**Caps Dispenser**). Apply slight pressure to click the cap into place and then pull forward. Turn the cap in the desired direction and remove the protective top. Squeeze the trigger slowly and smoothly to extrude **x-tra fil** directly into the cavity.

Light-curing

x-tra fil is specially designed for applying in layers of up to 4 mm. Higher fillings must be built up in layers and each layer cured separately with a polymerization lamp: 10 sec. when using lamps rated at 800 mW/cm² or higher. 20 sec. when using lamps rated at 500 to 800 mW/cm² Hold the light emission tip of the light-curing device as close as possible to the surface of the filling. Ensure that every section of the filling is fully exposed to the light. An additional polymerization of the accessible case from the oral or vestibular side is recommended. Remove metal matrix previously. If the distance is more than 5 mm, the curing depth may be smaller so that only that area actually illuminated by the light cone will be cured. Incomplete curing may lead to discoloration and pulpitis-like complaints.

Finishing

After removing the matrices, the filling can be trimmed and polished immediately while being cooled (e.g. fine or extra-fine grit diamonds, abrasive disks). To conclude the procedure, the margin of the filling or rather the entire tooth should be fluoridated.

Information/precautionary measures:

- Phenolic substances, especially preparations containing eugenol and thymol inhibit curing of filling composites. Avoid the use of zinc oxide/eugenol cements or other materials containing eugenol in combination with filling composites.
- Fillings exposed to occlusal loading should be examined at least once per year to ensure that any changes can be identified in time.
- Allow the caps to attain room temperature before use and apply with even, but not excessive, pressure.

Storage:

Store at temperatures between 39°F - 73°F (4°C - 23°C). If refrigerated, the material must be allowed to reach room temperature before use. To avoid exposure to light and possible polymerization, syringes and bonding material should be closed immediately after dispensing. Do not use after the expiry date.

PRODUCT ORDERING INFORMATION:

x-tra fil	
5gm syringe (universal shade)	REF 1740
x-tra fil Caps	
20 x 0.25gm (universal shade)	REF 1741

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children. For dental use only.

An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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VOCO GmbH